

Hydrofera Blue® Bacteriostatic Wound Dressing

APPLICATION INSTRUCTIONS



1

Remove Hydrofera Blue Bacteriostatic Dressing from the package and thoroughly moisten it with sterile saline or sterile water.

Once Hydrofera Blue Bacteriostatic Dressing is thoroughly moistened, squeeze out excess liquid.



2


Apply dressing to the wound. If the wound presents with surrounding erythema or rolled wound edges, the dressing should be larger than the wound to cover the periwound area. In other wounds, the dressing may be cut to fit the wound bed.



3

Cover Hydrofera Blue Bacteriostatic Dressing with a secondary dressing to prevent displacement and to maintain moisture content.

The secondary dressing selection is dependent on level of exudate. For example, if there is low exudate, a transparent film or impregnated gauze may be used. If there is heavy exudate, rolled gauze or a foam may be used.

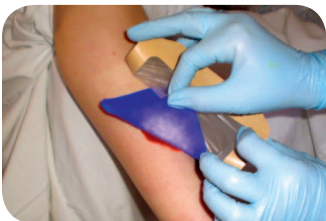

4

The first dressing change should occur at 24 hours. Examine the area of the dressing in contact with the wound bed. If the dressing has retained its blue color where it is in contact with the wound, apply a new dressing and leave in place for up to 72 hours. If the dressing has turned from blue to white or lightened in color, the dressing should be changed every 24 hours until it retains its blue color.

The dressing should be changed anytime there is exudate strikethrough.



5

Do not allow the dressing to completely dry out. Rehydrate the dressing as needed. If the dressing does dry out, thoroughly moisten it with sterile saline or sterile water, wait 2 minutes, then gently remove the dressing.

Hydrofera Blue®

Bacteriostatic Wound Dressings

Hydrofera Blue®

Bacteriostatic Wound Dressings

Secondary Dressing Selection Guide

Select a secondary dressing that will help maintain the moisture content of Hydrofera Blue® Dressing.

Level of Exudate	Secondary Dressing Examples
Low	Transparent Film Impregnated Gauze
High	Rolled Gauze Foam (e.g. Restore Hydro-Shield Dressing)

Ordering Information

Stock No	Product Size	Quantity/Box
Standard Dressings		
HB2214	2" x 2" (5cm x 5cm)	10
HB4414	4" x 4" (10cm x 10cm)	10
HB6614	6" x 6" (15cm x 15cm)	10
HBT0906	9mm Tunneling Dressing (1.2gm)	10 pair
Heavy Drainage Dressings		
HBHD4450	4" x 4" x .5" thick (10cm x 10cm x 1.2cm)	5
HBHD6675	6" x 6" x .75" thick (15cm x 15cm x 1.9cm)	5
Dressings with Moisture-Retentive Film		
HBRF4414	4" x 4" (10cm x 10cm)	10
HBRF2258	2.25" x 8" (5.7cm x 20cm)	10
HBRF4475	4" x 4.75" (10cm x 12cm) Island Dressing <i>pad size 2" x 2.75" (5cm x 6.5cm)</i>	10
HBRF2650	2.5" (6cm) Diameter Ostomy Dressing	10

Hydrofera®

Pioneering Advancements in PVA Technology

Hydrofera, LLC
322 Main Street
Willimantic, Connecticut 06226
Phone: 860.456.0677
Fax: 860.456.0898
E-mail: Hydrofera@aol.com

www.hydrofera.com